

HEADALOFT™

Installation and Maintenance Guidelines

HEADALOFT™ is a patented, innovative solution for adults and children who use wheelchairs.

HEADALOFT™ comfortably and gently holds the head upright in a versatile, non-restrictive manner.

- **HEADALOFT™** has an occipital support cushion that can be added to provide additional head and neck support, as needed.



Suitable for all wheelchairs equipped with a head rest and car seats, **HEADALOFT™** is the ultimate head support add-on solution. It is easy to install, can be transferred quickly between wheelchair and car seat or any other type of seat that has a headrest to which the **HEADALOFT™** can be attached.



Technical Specifications


- Made of high-quality, durable fabric.
- **HEADALOFT™** filling is flame-retardant.
- Includes three different size head support straps to enable it to be adjusted to any head size.
- Attaches to all wheelchairs equipped with headrests.
- Fits most car seats.
- crush tested under U.S. crash test safety standards US - FMVSS 213, NPRM 213.
- Provides 360° head support.
- Meets OKEOTEX 100 standard
- FDA and CE registered



Headovations warrants that for a period of 12 months from the date of purchase, the product will be free from material defects in material and workmanship, subject to Headovations policy and provided that the product shall have an original identification label. The above Warranty is subject to the buyer establishing that the product has been correctly handled and stored, in accordance with Headovations' instructions. In the event that Headovations finds that the product is defective, Headovations shall replace the damaged product.

Installation Instructions:

Installing **HEADALOFT™** on any seating and mobility device equipped with headrest is very easy and can be accomplished very quickly. Follow these simple instructions step-by-step to install **HEADALOFT™**.

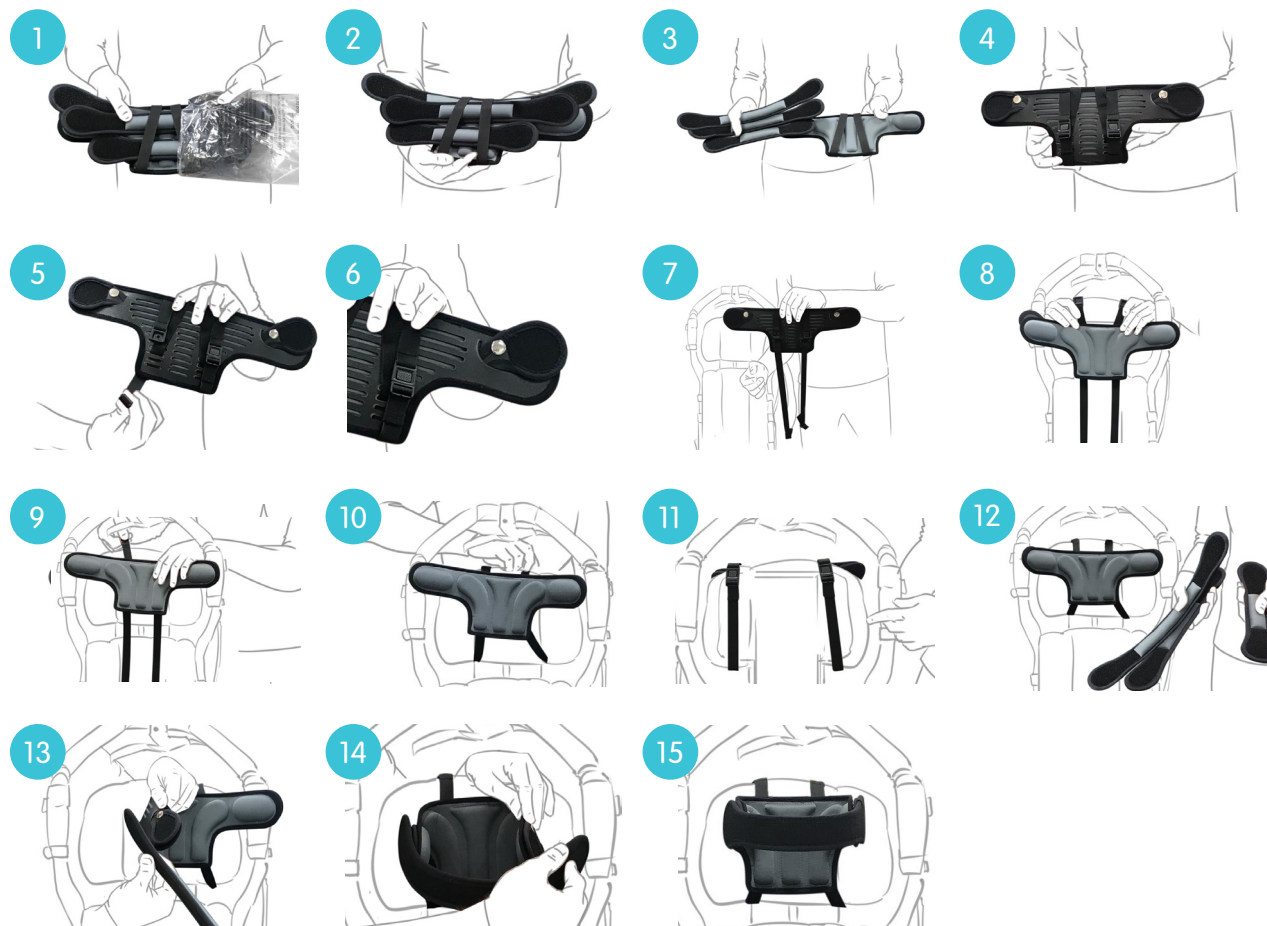

Maintenance:

HEADALOFT™ requires no real maintenance. It is extremely durable and requires only occasional washing. It can be machine washed in temperatures up to 30°C / 86°F, however you should not use bleach as this can damage the appearance and material.

After washing **HEADALOFT™**, you should not iron it or put it in the dryer. Instead, allow it to dry naturally. It should also not be dry cleaned.

Storage:

When not in use, **HEADALOFT™** should be stored in a clean, dry area.

Headovations was founded in 2014 by Ben Cohen Gazit. Dedicated to providing innovative head support systems.

HEADALOFT™ has been shown to be extremely effective for children and adults with motor disorders, neuromuscular disorders, neurological disorders (e.g. cerebral palsy), and physical injuries.

Developed by:



In Partnership with:

